


# Varios Combi Pro2



**NSK**



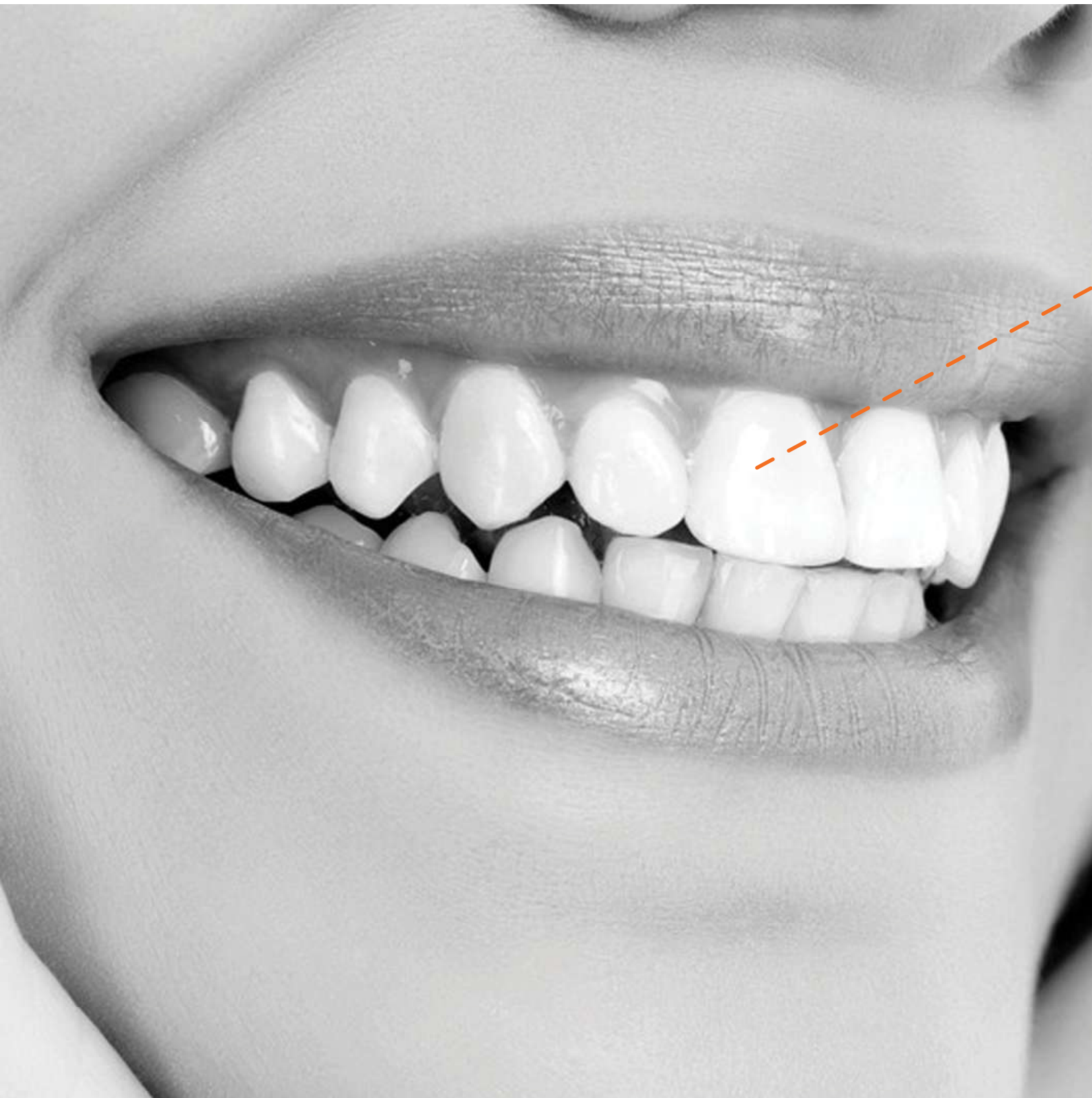

# 32

## reasons to buy the Varios Combi Pro2

After listening to the needs of a broad range of dental professionals - we've crafted a combi-device that delivers the advanced functionality clinicians really want.

Combining subgingival and supragingival powder therapy, with ultrasonic scaling for periodontal treatment, the Varios Combi Pro2 is designed for ease of use, adapting to your preferred treatment style.

Born from NSK's wide range of experience and research, the Varios Combi Pro2 provides the professional care you have come to expect from NSK. With its durability and effortless maintenance, it is your perfect 'daily driver', helping you perform routine dental work smoothly and efficiently - without interruptions.

With NSK by your side, you have the tools to perform at your very best - providing smooth, efficient care, from prevention to treatment, for every patient, every time.

## ULTRASONIC SCALER

### Easy-eject water bottle

For your ease and convenience, the water holder can be easily detached and reattached.



### Titanium slim scaler handpiece

Including our world-first ultrasonic scaler handpiece made from **pure titanium**. Our unique **DURAGRIP** finish provides excellent grip and stability during use, with the LED light providing great viewability.



### Adjustable water volume

With our analogue dial, you have precise and intuitive control of water volume.

Each dial corresponds to the ultrasonic scaler and powder functions.



### Water heating

Both the ultrasonic scaler and the powder handpiece will inject heated water ensuring greater patient comfort.

# PURSUING



## POWDER THERAPY



# USABILITY



NSK Varios2 Lux

# ULTRASONIC SCALER



## Ultrasonic scaler


### Made to last. Comfortable to use.

At NSK we do not compromise on quality. Our ultrasonic scaler handpiece offers great user-friendliness, comfort, durability, convenience, accessibility, and visibility.

Our Varios Combi Pro2 is equipped with our world-first ultrasonic scaler handpiece - made from pure titanium: heat and chemical resistant, and able to withstand multiple reprocessings and autoclave sterilisations.

In addition, to ensure great grip and stability, our scaler handpiece is finished with the highly regarded **DURAGRIP**, used in NSK's premium handpiece range.

This elegant, durable, well-balanced and lightweight scaler is comfortable to use, with long lasting performance.

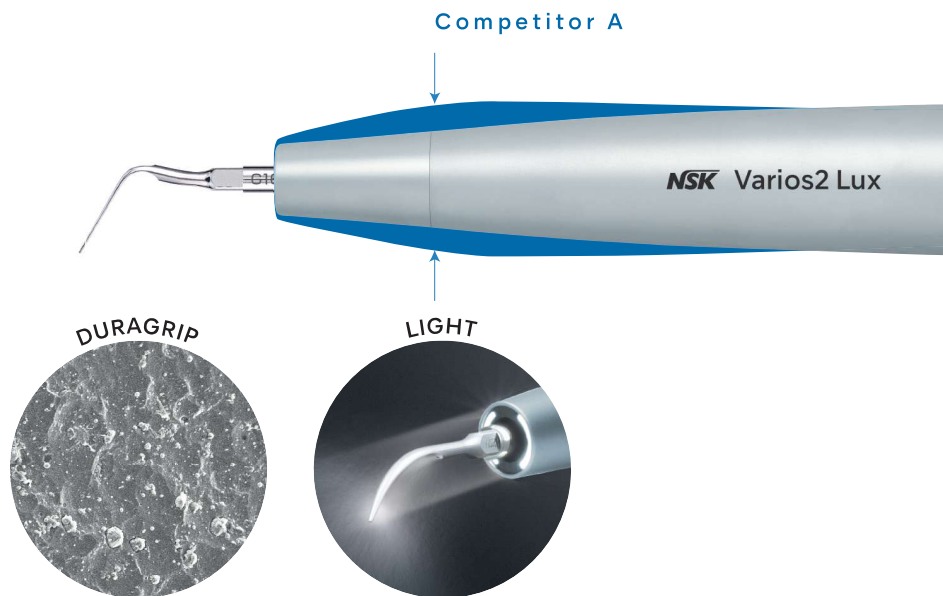



# POWDER THERAPY




# Powder therapy

## Slim, exchangeable nozzles.

Our powder handpieces and nozzles are designed to make your work easier and more efficient:

- Accessing the molar region is easier with the long, narrow nozzles.
- You'll experience excellent visibility and maneuverability within the oral cavity with the slim handpieces and nozzles.
- A wide operating field can be ensured on the lingual aspect without opening the lips wider than necessary on the buccal side.
- The nozzles are fully replaceable right at the chairside, making maintenance and repairs hassle-free.

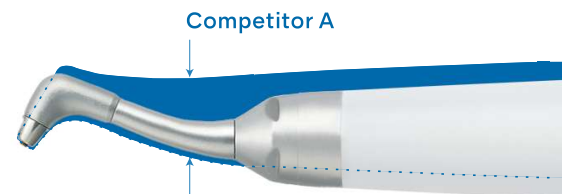

## Jet Nozzle for perio powder handpiece

In response to clinicians' demand, we've developed a Jet Nozzle for the Perio Handpiece - to be used in supragingival treatment. In addition to the previous Perio Nozzle, it is now possible to precisely target the appropriate area with this Jet Nozzle. These two types of perio nozzles can be interchanged depending on the desired application.




## Constant availability



### Independent unit & chambers

With the Varios Combi Pro2, all the powder guiding lines (powder chamber, hose, and handpieces) are separate from the main unit. In the unlikely event of powder clogging, the parts can easily be detached by hand. There is generally no need to dismantle or send the entire device to the supplier or manufacturer for repairs or part replacement. Your practice can therefore keep running and generating income - without interruptions.

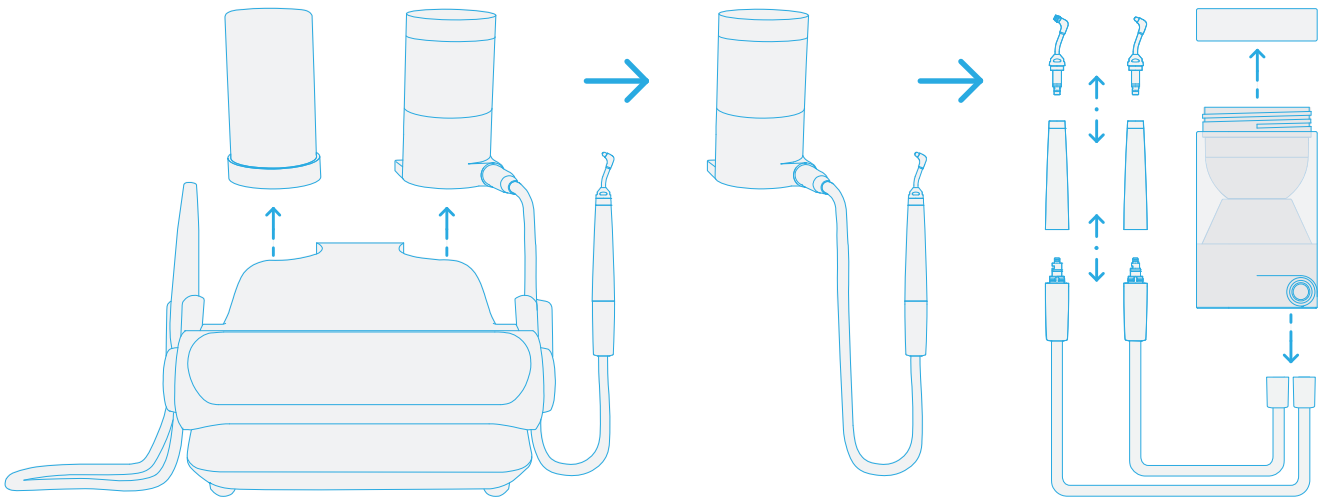

The main unit's interior features a design based on a smart concept: having air and water circuits only.



### No routine servicing required

As the Varios Combi Pro2 is designed for easy care and upkeep by you, regular maintenance by your local NSK service provider is not required. Practice downtime due to maintenance is therefore a thing of the past. Should the need arise, our friendly support team is always available.



## Easy care

### Auto-Cleaning



The Varios Combi Pro2, has four auto-cleaning modes that can be run (as appropriate) after each patient session, or at the end of the day.

These avoid the need for a multitude of tools, while reducing the risk of powder clogging and water accumulation. They also keep the inside of the device clean at all times.



Auto-cleaning  
Powder



Auto-cleaning  
Powder



Auto-cleaning  
Ultrasonic Scaler



Auto-cleaning  
Dual



### CLM mode



Cleaning the water circuit inside the unit is simple, using our b.clean solution and CLM mode setting, making the Varios Combi Pro2 compliant with European reprocessing requirements. As an additional built-in safety mechanism, the unit is set to disable regular use, until the chemical solution is fully drained.

## Patient comfort

### Water heating system




The Varios Combi Pro2 is equipped with a water circuit heater for both the ultrasonic scaler and the powder handpieces. With the temperature of the injected water being approximately the same as that of the oral cavity - greater comfort for a wide range of patients is ensured, even those with sensitivity.

# Operability

## Wireless multi-function foot-control

Position the foot-control where it is most optimal for you, without concern for any limiting cords / cable lengths.

The Varios Combi Pro2 foot-control connects via Bluetooth®, and enables carefree and convenient operation using its multi-function control options - so you can concentrate fully on the treatment at hand.

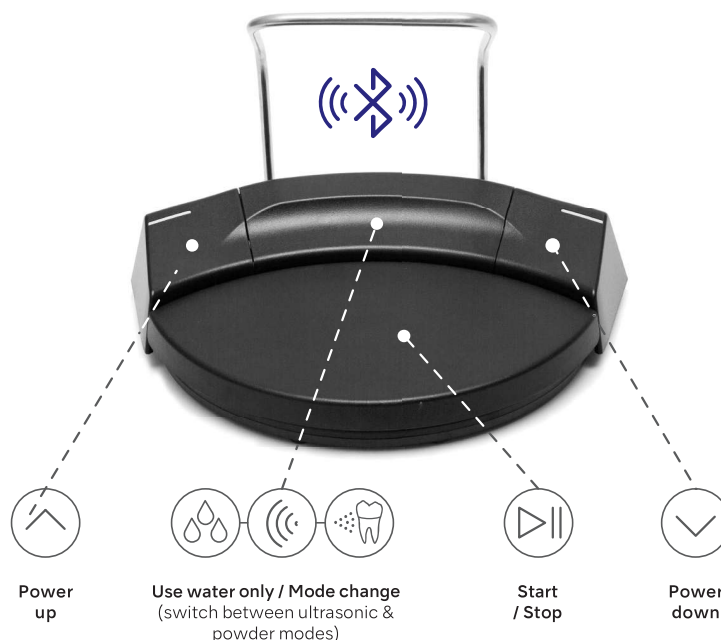

## iCart Prophecy2 with water bottle kit\*

Our specialised cart brings a new dimension of user-friendliness and convenience.


It can be fitted with an independent water-supply system; therefore, combining the Varios Combi Pro2 with the iCart Prophecy2, enables you to treat patients in environments where external water availability or quality is a concern.



\* Water bottle kit is optional

# Ultrasonic scaler tip variations

Our diverse range of more than 70 types of ultrasonic scaler tips, means that the ultrasonic scaler is suitable for a wide range of clinical applications, including both preventative care and the treatment of dental conditions.

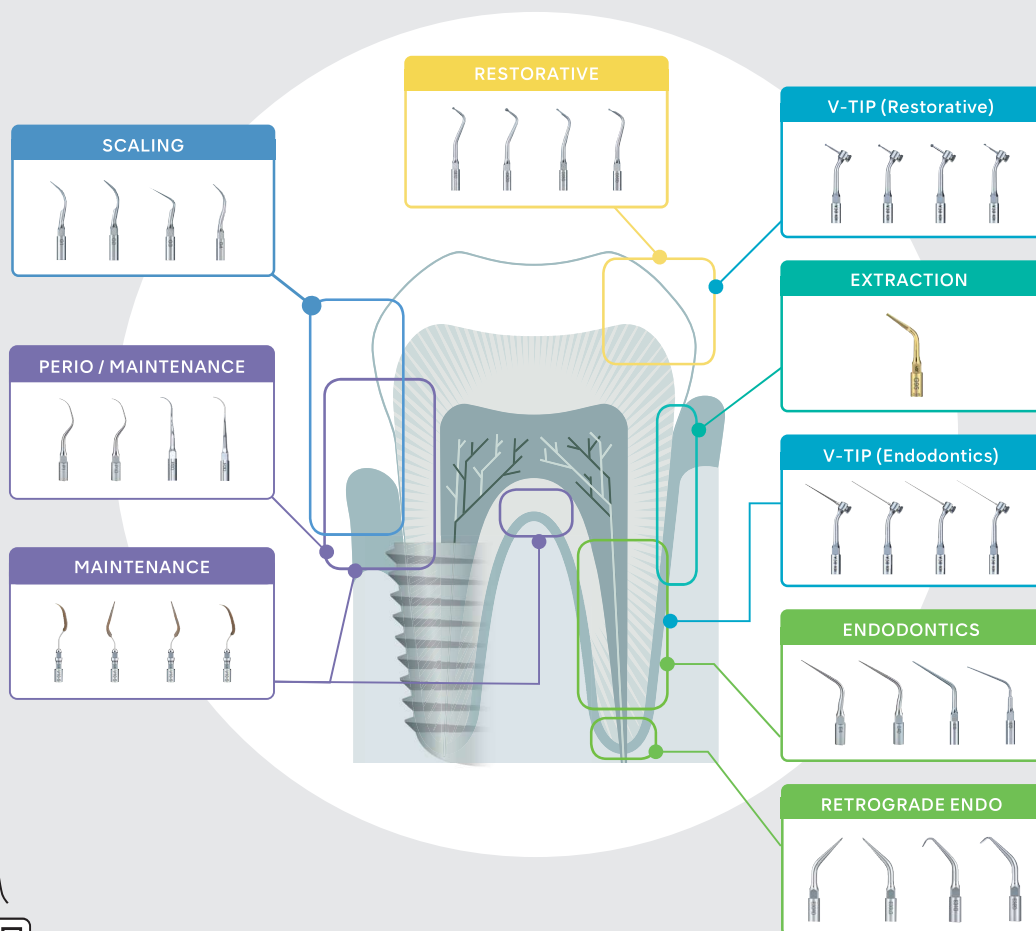

Scan to  
download  
Tip Guide

Link: [www.nsk-dental.com/products/oral-hygiene/oral-varios\\_ultrascaler\\_tips/](http://www.nsk-dental.com/products/oral-hygiene/oral-varios_ultrascaler_tips/)



# POWDERS

# Powders

## Maintaining healthy and beautiful smiles

Powder therapy effectively eliminates subgingival and supragingival stains and biofilm. By utilising the Varios Combi Pro2 benefits and protocols, you can unlock a new era of professional care - delivering exceptional results. Care-free for you, and reassuring for your patients.

### FLASH pearl

Polishing powder for supra-  
gingival use, with spherical  
particles that are gentle on  
the tooth surface.

**Intended use:** removal of  
stubborn stains

Principal ingredient: Calcium  
carbonate

- contains no sodium chloride  
(no salty taste)
- suitable for patients on a  
restricted sodium diet

Average particle size: **54µm**



MODEL	ORDER CODE
<b>FLASH Pearl</b> 300g bottle x 4	<b>Y900698</b>

### SOFT pearl

Polishing powder for supragingival  
use with low surface invasiveness.  
Suitable for use even with some  
prosthetic devices.

**Intended use:** removal of  
biofilm and moderate stains

Principal ingredient: Glycine

- unflavoured
- sweet taste

Average particle size: **65µm**



MODEL	ORDER CODE
<b>SOFT Pearl</b> 250g bottle x 4	<b>Y1500721</b>

### Perio Mate Powder


Polishing powder for gingival  
margin and subgingival use.

**Intended use:** removal of biofilm  
and minor staining

Principal ingredient: Glycine

- unflavoured
- sweet taste
- highly water-soluble, so does  
not accumulate even when  
used subgingivally



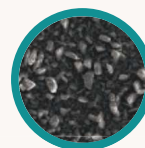





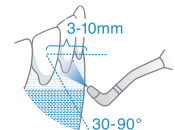





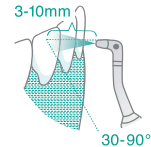



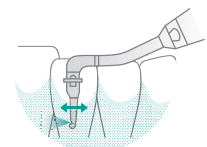
Average particle size: **25µm**

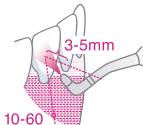

MODEL	ORDER CODE
<b>Perio Mate Powder</b> 160g bottle x 2	<b>Y900938</b>

# Powder therapy

## Powder + Nozzle combinations & uses

MODE	POWDER KIT	HANDPIECE	NOZZLE	USE	POWDER
<b>PROPHY MODE</b> 	<b>FOR PROPHY</b> 				
					
<b>PERIO MODE</b> 	<b>FOR PERIO</b> 				
					



# MIT

## Minimally Invasive Therapy

With the Varios Combi Pro2 you can deliver NSK's Minimally Invasive Therapy (MIT)  
- helping you, the oral hygiene hero, to be your best version.

Complete STEP 1, STEP 2, and STEP 4 of the MIT protocol, using its ultrasonic scaler and  
powder therapy, for supragingival and subgingival treatment.



NSK is committed to quality oral hygiene and creating intuitive, daily driver dental products.

#### SOURCES & REFERENCE:

Herrera, David, Mariano Sanz, Moritz Kebschull, et al. 2022. "Treatment of Stage IV Periodontitis: The EFP S3 Level Clinical Practice Guideline." Journal of Clinical Periodontology 49 Suppl 24 (S24)

# Varios Combi Pro2

## Basic set



OPTIC	MODEL	ORDER CODE
●	Varios Combi Pro2 Basic Set	Y1500505
<div>Contents:</div> <ul style="list-style-type: none"><li>Varios Combi Pro2 Control Unit</li><li>VA2-LUX-HP Ti</li><li>VA2-LUX-SC Ti</li><li>Prophy-Chamber2</li><li>Prophy-Hose2</li><li>Powder Handpiece (Prophy)</li><li>Wireless Foot Control</li><li>Hanger</li><li>Air Hose</li><li>Water Hose</li><li>Irrigation Bottle Set</li><li>CLM Bottle Set</li></ul> <div>Specifications:</div> <ul style="list-style-type: none"><li>Dimensions: W 249 x D 290 x H 115 mm (including chambers)</li><li>Weight: 3.5 kg</li><li>Power source: AC 220 - 240 V 50/60 Hz</li><li>Drive frequency (ultrasonic): 28 - 32 kHz</li><li>Maximum output (ultrasonic): 11 W</li></ul>		

## Optional



MODEL	ORDER CODE
Prophy Kit2	Y1500469
<div>Contents:</div> <ul style="list-style-type: none"><li>Prophy-Chamber2</li><li>Prophy-Hose2</li><li>Powder Handpiece (Prophy)</li><li>FLASH pearl powder (300g)</li></ul> <div>* Included in the Varios Combi Pro2 Basic set.</div>	



MODEL	ORDER CODE
Perio Kit2	Y1500470
<div>Contents:</div> <ul style="list-style-type: none"><li>Perio-Chamber2</li><li>Perio-Hose2</li><li>Powder Handpiece (Perio)</li><li>Perio Nozzle Tip 40pc</li><li>Perio Mate Nozzle tip remover</li><li>Perio Mate Powder (160g)</li></ul>	

## Optional items

MODEL	PRODUCTS	ORDER CODE
<b>iCart Prophy2</b> Dimensions: H 840 mm Weight: 16 kg		<b>S9098</b>

MODEL	PRODUCTS	ORDER CODE
<b>VA2-LUX-HP Ti</b>		<b>EA1203X050</b>

MODEL	PRODUCTS	ORDER CODE
<b>B.clean</b> (1L bottles x 6)		<b>BCL600</b>

MODEL	PRODUCTS	ORDER CODE
<b>Water Bottle Kit</b> Contents: • Irrigation Bottle Set • CLM Bottle Set • Purge Tool		<b>Y1500719</b>

MODEL	PRODUCTS	ORDER CODE
<b>Powder Handpiece</b> (for Prophy use)		<b>T1074001</b>

MODEL	PRODUCTS	ORDER CODE
<b>CLM Module Set</b> Contents: • CLM Perio attachment • CLM Prophy attachment		<b>Y1500740</b>

MODEL	PRODUCTS	ORDER CODE
<b>Powder Handpiece</b> (for Perio use)		<b>T1077001</b>

## Spare parts

MODEL	PRODUCTS	ORDER CODE
<b>Nozzle</b> (for Prophy use)		<b>T1074012</b>

MODEL	PRODUCTS	ORDER CODE
<b>Irrigation Bottle Set</b> (for Water Bottle kit)		<b>ZA16160001</b>

MODEL	PRODUCTS	ORDER CODE
<b>Nozzle</b> (for Perio use)		<b>T1077012</b>

MODEL	PRODUCTS	ORDER CODE
<b>CLM Bottle Set</b> (for Water Bottle Kit)		<b>ZA16240001</b>

MODEL	PRODUCTS	ORDER CODE
<b>Jet Nozzle</b> (for Perio use)		<b>TA10770106</b>

  
Washable in the  
thermodisinfectant

  
Autoclavable

**NAKANISHI INC.** [www.nsk-dental.com](http://www.nsk-dental.com)  
700 Shimohinata, Kanuma  
Tochigi 322-8666, Japan

**NSK Europe GmbH** [www.nsk-europe.de](http://www.nsk-europe.de)  
Elly-Beinhorn-Str. 8  
65760 Eschborn, Germany

**NSK United Kingdom Ltd.** [www.nsk-uk.com](http://www.nsk-uk.com)  
Office 4, Gateway 1000, Arlington Business Park  
Whittle Way, Stevenage, SG1 2FP, UK

**NSK Dental Nordic** [www.nsk-nordic.com](http://www.nsk-nordic.com)  
Furstenbergsgatan 4  
416 64 Gothenburg, Sweden

Specifications are subject to change without notice.

Create it 