

Score saddle chairs:



Score Saddle chairs: advantages

The Score Dental saddle stool has three major advantages:

1. The natural upright sitting posture prevents back and neck injuries.

Traditional dental stools facilitate only a limited hip angle. This makes it very tiring to keep your spine, neck, and head in a favourable vertical alignment. Staying seated upright then simply costs too much muscle strength. The result is that after a while you will automatically slouch and even start working in a bent over position. Score Dental saddle stools prevent this. The unique shape of the saddle, and the elevated sitting height open up the hip angle (up to 140 degrees) when your pelvis is tilted forward, automatically placing your spine, neck and head in a healthy vertical alignment ('neutral posture'; see below). You maintain a healthy standing-like posture, while also enjoying the beneficial stability and comfort of traditional sitting.

2. You can sit closer to the patient. No need to bend over as much.

With a hip angle of 90 to 110 degrees, your legs are often in the way of you sitting close to your patient, so you bend your back more and put strain on it in order to gain a good view into the mouth. Adjusting the treatment chair higher is not a solution because then you have to lift your arms higher, unwantedly increasing the tension on your shoulders and neck. On a Score Dental saddle stool, you sit in an elevated position, with your upper legs oriented in a more downward and sideways direction (instead of forwards). This allows you to better position your legs under the treatment chair, allowing you to sit much closer to the patient. Moreover, you can move around your patient easily because your legs are somewhat spread and you can push off to left and right easily.

3. Your open posture ensures better concentration and increased energy.

Prolonged static sitting on a traditional chair can lead to a dull or tingling feeling as blood vessels are often pinched. The open posture the ergonomic shape of the Score Dental saddle stool puts you in makes breathing easier, so that you take in more oxygen. In addition, digestion improves, and blood circulation is optimised. The result? Better concentration, greater focus, and a more active posture helping you have more energy throughout the entire day so that you can help your patients better.

Score saddle chairs: mechanisms

Chairs mechanisms: Two options: Balance system or Adjustable seating angle (Tilt)

Balance (Dynamic sitting)

The Score Dental Balance mechanism ensures maximum comfort and optimal dynamic working posture. This adjustable mechanism enables the saddle to move subtly along with you for extra comfort and a beneficial effect on the muscles and spine. The uniquely adjustable Balance mechanism from Score Dental ensures that the saddle moves subtly along with your movements. These 'micro' movements of your pelvis ensure that muscle groups and intervertebral discs are not constantly under severe static strain. This prevents pain, helps you maintain your working posture longer, and exercises your core muscles. The degree of movement can be adjusted by rotating the ring.

Adjustable seating angle (Tilt)

Tilting seat Instead of the Balance mechanism, you can also choose a tilting seat where the angle of the seat can be adjusted forwards and backwards.



Extra support: Lumbar Support adjustable in height and angle

Score Dental saddle stools can be equipped with a lumbar support. A lumbar support that is adjustable in both height and angle. Its multifunctional shape provides light support for your lower back (lumbar area) while working in an active posture. In addition, you can also lean against it and stretch to relax your back for a moment.



Score saddle chairs: How to sit correctly?

Correct height.


You will only experience the full benefits of the saddle stool if your position is high enough to sufficiently open up your hip angle, thereby placing you in a natural and relaxed posture. The perfect sitting height is the one where your upper legs naturally follow the shape of the seat. This enables the maximum hip angle to be achieved (130 to 140 degrees) while keeping your weight evenly divided across the seat of the saddle stool and the floor. Because people vary in height, Score Dental saddle stools are available in three different heights, each with its own range of height adjustment. For nine out of ten people, the standard height provides a correct adjustment range to position the stool at the optimal height. It is therefore the most popular option if the stool is going to be shared by multiple persons.



Score saddle chairs: Amazone or Jumper seat?

Perfect fit

Score Dental saddle stools are available in two unique seat shapes. The Amazone for an active narrow sitting posture, and the Jumper for a wider sitting posture.

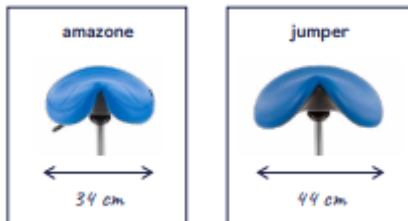

Amazone: The shape of the Amazone's seat facilitates the most open hip angle. Your weight is divided evenly across the seat and the floor. The Amazone seat strikes the best possible balance between the control and healthy posture of 'standing', and the comfort of 'sitting'.

Jumper The Jumper has a wider seat, resulting in a slightly smaller hip angle than the Amazone. A larger part of your body weight rests on the seat. This gives the Jumper seat a more traditional sitting experience.

Score saddle chairs: height of the gas lift.

Score saddle chairs can be equipped with different gas lifts for different heights.

Heights	Height adjustment		Work surface height
Low	MID25	app. 49 - 63 cm	70 - 85 cm
Standard	MID31	app. 59 - 77 cm	80 - 100 cm
High	MID38	app. 63 - 88 cm	90 - 115 cm
*Saddles with Balance mechanism (B/T) + app. 3 cm			
*Saddles with lumbar support (B/T) + app. 3 cm			

Seat height Score Amazone	Adjustable seating angle or Balance with lumbar support	Fixed or adjustable seating angle without lumbar support
Low (ca. 49 - 63 cm)	< 165 cm	< 160 cm
Standard (ca. 59 - 77 cm)	160 - 185 cm	155 - 180 cm
High (ca. 63 - 88 cm)	> 170 cm	> 165 cm

Seat height Score Jumper	Adjustable seating angle or Balance with lumbar support	Fixed or adjustable seating angle without lumbar support
Low (ca. 49 - 63 cm)	< 170 cm	< 165 cm
Standard (ca. 59 - 77 cm)	165 - 190 cm	160 - 185 cm
High (ca. 63 - 88 cm)	> 175 cm	> 170 cm

Score saddle chairs: Upholstery choices: Stamskin or PUxx

Main difference: Stamskin cannot be cleaned with alcohol-containing disinfectants, whereas PUxx can.

With or without seams:

Please note that the Stamskin upholstery can be with seams or seamless. PUxx upholstery can only be seamless.

STAMSKIN

Composition: Stamskin TOP F 4340, backing Polyamide-jersey, coating antimicrobial and anti-fungal treatment

Weight: 780 g/m²

Light fastness: Scale 7

Abrasion resistance: > 120.000 rubs Martindale

Advantages: Urine-, blood-, sweat, fungal and waterproof, manufactured to environmentally friendly standards ISO 14001

Cleaning: With brush, water and soap, rinse off with water, do not clean with stainremovers containing solvents or oily products

N°528/2012/CE-Folpet: Stamskin Top contains the biocidal product Folpet, an antifungal additive used to help maintaining appearance, physical properties and prolong the service of these articles.

Flame retardant:

UK: BS 5852	D: DIN 53438/T2	F: NFP 92.507
-------------	-----------------	---------------



PUxx

Composition: PUxx no. 1, 100% Polyurethane, backing polyester cotton mixture, free from Phthalate and PVC.

Weight: ± 520 g/m²

Light fastness: 6-7 (8 is excellent)

Abrasion resistance: > 300.000 Martindale

Advantages: PUxx nr. 1 is free of Phthalate and PVC and OEKO-TEX certified. 100% waterproof. Urine and blood resistant. Extremely scratch resistant surface. Easily transport for heat (no sweat feeling).

Cleaning: PUxx no. 1 is easy to clean and disinfect. It is resistant to most cleaning agents and disinfectants (based on alcohol, chlorine and ammonia). Do not use solvents.

Flame retardant:

GB BS 5852 Crib 5	EN 1021 1-2	CALL117-2013	IMO 2010 FTP: part 8	D DIN 4102 B1 (available on request only)
-------------------	-------------	--------------	----------------------	--



Stamskin (with seams or seamless):

Official website Stamskin Top with all colours (also non-standard), specifications etc.:

<https://www.sergeferrari.com/productsbatyline-range/stamskin-top>

Stamskin Top brochure:

<https://sergeferrari.sharepoint.com/sites/WEBSITEPI/Brochures/Sample-Card-Batyline-Stamskin-Top-Club-EN-ES.pdf?ga=1>

Link to tested cleaning disinfectants for Stamskin Top:

<https://www.medizina.de/media/content/Polster-Pflegehinweise/StamskinTop-Desinfektionsmittelempfehlung.pdf>

PUxx (only seamless):

Official website PUxx with all colours (also non-standard), specifications, certifications, etc.: <https://www.onirotextiles.com/puxx-nr.-1-uk.html>


Link to the upholstery card: <https://www.onirotextiles.com/puxx-color-card.html>

Link to the cleaning instructions:


<https://www.onirotextiles.com/assets/uploads/homepagina/Puxx/Cleaning%20instructions%20PUxx%20nr.%201%20+%20nr.%202.pdf>

Score saddle chairs: Steps of configuration:


1. Seat: Amazone or Jumper

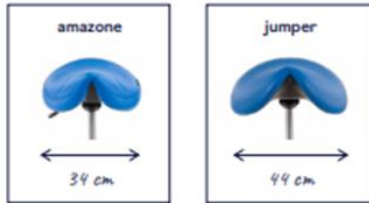


2. Mechanism: Adjustable seat angle or Balance

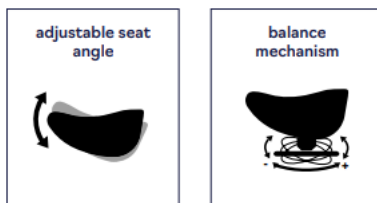

3. Without or with adjustable lumbar support

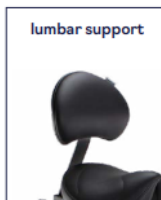

4. Upholstery Stamskin or PUxx, with seams or seamless

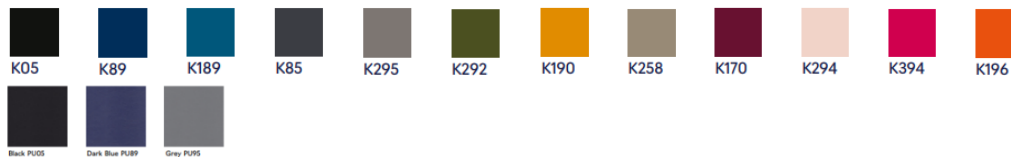


5. Height of gas lift: standard, low or high



- Low (MID25)
- Standard (MID31)
- High (MID38)

6. Starbase: standard of small

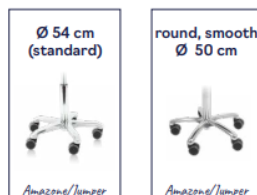

7. Extra options:

Height adjustment by foot control



360° support (rotative armrest)



Score saddle specials:

Score Amazone Balance Alba and Jumper Balance Alba



Please note: the Score Balance Alba models are only available with a Balance mechanism and without lumbar support.

Starbase and gas lift can be in white or aluminium. Castors are white.

The seat can be upholstered in Stamskin colours.

Score saddle chairs: Marketing

Photos and images in Score marketing portal:

Score saddle chairs photos in high and low resolution:

<https://scoremarketingportal.stackstorage.com/s/ScoreDealer/nl/files/5359>

Amazon saddle chairs High resolution:

<https://scoremarketingportal.stackstorage.com/s/ScoreDealer/nl/files/81401>

Jumper saddle chairs High resolution:

<https://scoremarketingportal.stackstorage.com/s/ScoreDealer/nl/files/81342>

Upholstery colours:

<https://scoremarketingportal.stackstorage.com/s/ScoreDealer/nl/files/8304>

Logos:

<https://scoremarketingportal.stackstorage.com/s/ScoreDealer/nl/files/40389>

Score Balance Alba models High resolution:

<https://scoremarketingportal.stackstorage.com/s/ScoreDealer/nl/files/42709>